



# Rapid Test Kits

Orient Gene COVID-19 Nasal Rapid Antigen Self-Test Kit



## Secure your order

Freephone: 0800 864 725

sales@unipak.co.nz

www.bastionpacific.co.nz

# Orient Gene Rapid Test

- ✓ Tested for COVID-19 Omicron
- ✓ Ministry of Health approved
- ✓ 98.74% accuracy



## Overview

- Facilitates patient treatment decisions quickly
- Rapid test for SARS-CoV-2 antigen within 15 minutes
- Simple, time-saving procedure
- All necessary reagents provided and no equipment needed
- High accuracy, sensitivity and specificity
- Certifications: CE, MSDS, ISO13485

## Ministry of Health Approval

The Orient Gene Rapid COVID-19 (Antigen) Test by Zhejiang Orient Gene Biotech Co., Ltd (China) is authorised under the COVID-19 Public Health Response 2020 (Point of Care) Order.

These have gone through a thorough evaluation and are approved for import, supply and use in Aotearoa New Zealand by the Ministry of Health.

[Click here](#) to view the Ministry of Health Rapid Antigen Testing approval.



# Rapid Test Kits

## Orient Gene COVID-19 Nasal Rapid Antigen Self-Test Kit



### Secure your order

Freephone: 0800 864 725

[sales@unipak.co.nz](mailto:sales@unipak.co.nz)

[www.bastionpacific.co.nz](http://www.bastionpacific.co.nz)

### About the Orient Gene Rapid Test

The Orient Gene COVID-19 Rapid Test antigen kit is a rapid diagnosis product that offers great reliability, and allows immediate detection of coronavirus antibodies 24-48 hours after infection with a nasopharyngeal sample.

This new Rapid Test is produced by Orient Gene Biotech, one of the largest Vitro test manufacturers in the world, is a CE certified diagnostic product, and has been recognized as an IVD medical device.

The Orient Gene Rapid Test kit is an immunochromatographic assay for the qualitative detection of the SARS-CoV-2 nucleocapsid protein antigen in nasopharyngeal samples from individuals suspected of having COVID-19. Helps quickly diagnose SARS-CoV-2 infections in just 15 minutes.

These test kits are intended for self-testing purposes by untrained individuals outside of laboratory or clinical settings such as in your home. You can use this test if you have symptoms of COVID-19 infection, particularly within the first 7 days following the start of symptoms.

### Technical Information

Package Size	5 tests per pack, minimum order quantity: 100 tests / 20 packs
Pack includes	Test cassettes Sterile swabs Extraction tubes and dropper tips Workstation Buffers Package insert
Collection type	Nasal, Nasopharyngeal, or Oropharyngeal swab
Results time	A qualitative result can be read after 15 minutes
Accuracy (Overall)	98.74%
Sensitivity	96.72%
Specificity	99.22%
Storage	The kit can be stored at room temperature or refrigerated Range of 2-30 degrees Celsius
Shelf Life	24 months



# Rapid Test Kits

Orient Gene COVID-19 Nasal Rapid Antigen Self-Test Kit



**Secure your order**


Freephone: 0800 864 725

sales@unipak.co.nz

www.bastionpacific.co.nz


## Instructions for use

### Specimen Collection & Sample Preparation

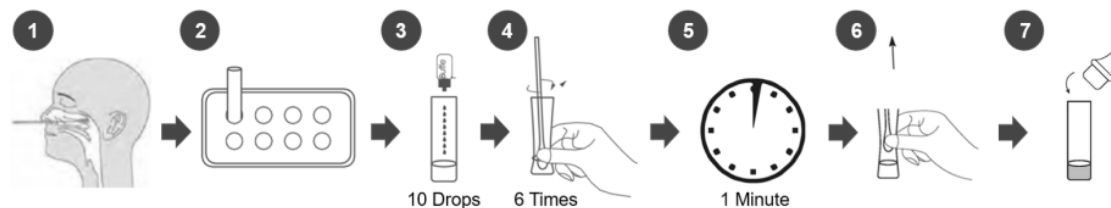

1. Insert swab into nostril of patient and rotate around the surface of the posterior nasopharynx.
2. Insert the extraction tube into the workstation.
3. Add 0.3 mL (10 drops) of the sample extraction buffer into the extraction tube.
4. Insert the swab into the extraction tube and roll it within the buffer at least 6 times .
5. Leave the swab in the extraction tube for 1 minute.
6. Squeeze the tube several times with fingers from the outside to immerse the swab.
7. Remove the swab and fit the dropper tip with filter on top the extraction tube tightly.

### Test Procedure & Results Interpretation - Read here for more information

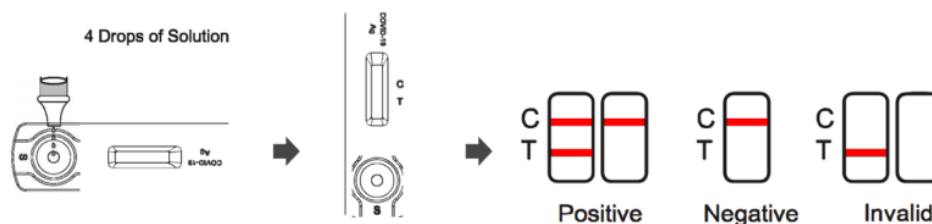

## Accredited by

Validated By:



Registered With:

